IUCC02630ITEN.03 - 01/07/2024 Pag. 1 a 1

Xylose Control Set

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This Values Table is applicable to sublots identified by the sequential alphabetical letter (for example A, B, C, etc.) following the lot number.

COMPONENTE / METODO COMPONENT / METHOD	VALORE VALUE	INTERVALLO RANGE	UNITÀ UNIT
Test enzimatico U.V. per la determinazione dello Xilosio Enzymatic UV test for the determinations of Xylose			
Controllo Livello 1 Control Level 1			
D140.25A 2026/07	6.58	5.26 – 7.89	mg/dL
Controllo Livello 2 Control Level 2			
E140.25A 2026/07	64.44	51.55 – 77.33	mg/dL

INTENDED USE

The device is used as a quality control to test the accuracy and precision of the quantitative determinations of Xylose in serum and urine. The device is intended for professional use only. For in vitro diagnostic use.

Concentrations and activities of components are studied for an excellent use even in automatic system of clinical chemistry, even in manual procedures.

PRESERVATION AND STABILITY

Preserve the product at 2-8°C, stable until expiration date on the package.

Once opened controls are stable for 15 days at 2-8°C.

MATERIALS REQUIRED BUT NOT SUPPLIED

Automatic micropipette, normal laboratory equipment.

REAGENTS PREPARATION

Bring controls at room temperature before use.

The controls are supplied in liquid form.

PRECAUTIONS

- Product contains Sodium Azide 0.095% as preservative.
- Avoid contact with skin and mucous membrane.
- Use the normal precautions for lab behaviour.

DISPOSAL CONSIDERATIONS

Product is intended for professional laboratories. Waste products must be handled as per relevant security cards and local regulations.

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LTA Srl