

IODIDE – CONTROL SET

Control solutions for determination of Iodide in urine

INTENDED USE

Concentrations and activities of components are studied for an excellent use even in automatic system of clinical chemistry, even in manual procedures.

PRESERVATION AND STABILITY

Preserve the product at 2-8°C, stable until expiration date on the package.
Once opened controls are stable for 7 days at 2-8°C.

MATERIALS REQUIRED BUT NOT SUPPLIED

Normal laboratory equipment.

REAGENTS PREPARATIONS

Controls are ready to use.
Bring controls at room temperature before use.

PRECAUTIONS

- Product contains Sodium Azide 0.095% as preservative.
- Avoid contact with skin and mucous membrane.
- Use the normal precautions for lab behaviour.
- For in vitro use only.

WASTE DISPOSAL

Product is intended for professional laboratories. Waste products must be handled as per relevant security cards and local regulations.

PACKAGING

CODE CC02830

Control level 1 1 x 2 ml (liquid)
Control level 2 1 x 2 ml (liquid)

REFERENCES





Archives data.

CALIBRATION VALUES

 E069.26*  2030/03

This Table of valueset for lot is applicable to sublots identified by the sequential alphabetical letter (for example A, B, C, etc.) following the lot number.









COMPONENT / METHOD	VALUE	RANGE	UNIT
--------------------	-------	-------	------

Colorimetric test for determination of Iodide in urine			
CONTROL LEVEL 1			
 H062.26A  2030/03	31.5	25.8 – 37.1	µg/l
CONTROL LEVEL 2			
 I062.26A  2030/03	231.0	189.4 – 272.5	µg/l

MANUFACTURER

LTA s.r.l.
Via Milano 15/F
20060 Bussero (Milan) ITALY
Tel: +39 02 95409034
Fax: +39 02 95334185
e-mail: info@ltaonline.it
Website: www.ltaonline.it

SYMBOLS

-  Only for IVD use
-  Lot of manufacturing
-  Code number
-  Storage temperature interval
-  Expiration date (year – month)
-  Warning, read enclosed documents
-  Read the directions
-  Biological risk

Mod. 01.06 (ver. 1.0 - 17/04/2012)

